



**sealtight**  
FREEDOM  
*The easy durable way to stay dry™*

REF 28004  
**ADULT KNEE/ELBOW**  
One (1) 19-23"  
Protector

# FREEDOM

**KNEE/ELBOW**

## Wound Protector Sleeve

Keeps injuries, bandages and PICC lines dry

<b>Easy to apply</b> Facil de aplicar Facile à appliquer	<b>Reusable</b> Reutilizable Réutilisable	<b>Guaranteed to perform</b> Garantizado para realizar Garanti pour effectuer
--	---	---

Seal-Tight® products are a part of BrownMed, Inc.



# Freedom Knee/Elbow Sleeve

SKU: 4246DEB8

Price: \$21.99

Category: [Medical Supplies](#)



## Instructions for Use

1. After removing your cast protector from the box, unfold completely.
2. Pull SEAL-TIGHT® Freedom over cast by stretching diaphragm opening with thumbs. DO NOT apply by pulling on rubber diaphragm.
3. Push the diaphragm toward the cast so the inner edge of rubber points up and away from cast.
4. To Remove: Dry rubber diaphragm and stretch diaphragm to ease over heel or elbow. DO NOT remove by pulling on the bag.

